Advazorb® Heel

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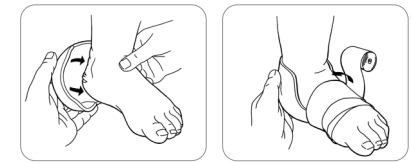
Utylizacja

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Advazorb® Heel

Hydrophilic foam dressing with film backing

Instructions for use

Advazorb® Heel

Hydrophilic foam dressing with film backing

Product Description

Advazorb® Heel is a conformable, non-adherent, hydrophilic, polyurethane foam dressing with a breathable film backing designed to fit around the heel.

A non-adherent foam dressing with excellent fluid handling properties. Ideal for use under compression bandaging with its fluid retention, low profile and low friction film backing.

Indications

Suitable for acute and chronic wounds including: cuts and abrasions, superficial burns, surgical wounds, leg ulcers, pressure ulcers and diabetic ulcers.

Contra-indications

Arterial bleeds and heavily bleeding wounds.

Directions for use

Apply directly to the wound on the heel, pink side up and secure in place with tape, appropriate bandage or secondary dressing.

Frequency of change

Can be left in place for up to 7 days but should be changed immediately if strike through is apparent or when dressing reaches capacity. If exudate is visible around the edges of the dressing this is a clear indication that a dressing change is required. Clinical observation is necessary to determine required frequency of change.

Warnings and Precautions

Advazorb® Heel is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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