Scan for more information

**Advancis**  
Medical

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# Advazorb® Border

# Advazorb® Border Lite

**Absorbent foam dressing with soft silicone wound contact layer and border**

*Instructions for use*

**Advancis**  
Medical

**Advazorb® Border/Advazorb® Border Lite** **EN**  
*Absorbent foam dressing with soft silicone wound contact layer and border*

#### Product description

A self adherent absorbent dressing with a Silfix® soft silicone wound contact layer and border.The dressing has a hydrophilic foam absorbent pad with high fluid handling capability.The soft silicone will adhere to surrounding dry skin but not to a wet wound. This is an atraumatic dressing which means it is designed to minimise the pain and trauma associated with dressing change.The film backing is breathable and prevents strike through.

#### Indications

A wide range of exuding wounds including: Leg, foot and pressure ulcers, traumatic wounds, surgical wounds, abrasions and blisters.

#### Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not use if allergic to silicone.

#### Directions for use

Remove the clear plastic liners and place directly onto the wound ensuring the absorbent pad covers the entire wound area with an overlap of 1-2cm. Can be used under compression. Secure in place with tape, appropriate bandage or film dressing.

#### Frequency of change

The dressing can be left in place for up 7 days but should be changed when exudate is visible in the centre pad or the pad swells due to absorbed exudate.

#### Warnings and Precautions

Advazorb® Border is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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