ADVANCIS

Brightwake Ltd, Lowmoor Business Park, Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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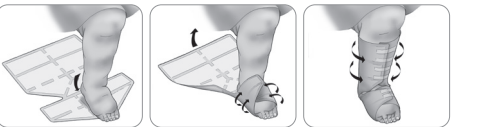
Super Absorbent Leg Wrap Dressing

Instructions for use

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI) carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	Device should not be cut		Moderate/High Exudate

Eclipse® Boot

Superabsorbent leg wrap dressing

Product description

Eclipse® Boot is an exudate management wound dressing especially designed for the lower leg. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds and lymphatic legs.

Contra-indications

Arterial bleeds and heavy bleeding wounds.

Directions for use

Open out the dressing fully, white side up placing the heel onto the small triangle, the top of the triangle pointing towards the patient. Fold over the foot one at a time the smaller sections to the left and right and secure with tape. Fold both the larger sections one at a time around the leg also securing with tape. Eclipse® Boot can be used under compression bandaging. Eclipse® Boot can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® Boot must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Warnings and Precautions

Eclipse® Boot is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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