

# Eclipse® Foot

## Super absorbent foot wrap dressing

Fact sheet


### Product description

Eclipse® Foot is a super absorbent dressing, designed to be applied securely around the foot area to manage wound exudate.

### Use

Open out the dressing fully, white side up placing the end of foot in the middle of the dressing so the heel is at the edge. Fold the side panels over the foot one at a time and secure with tape. Eclipse® Foot can be used under compression bandaging

Wear time will depend on the level of exudate, daily changes may be required, but Eclipse® Foot can be left in place for up to 7 days. Because of the excellent fluid handling capability of the dressing, it may become heavy and require changing.

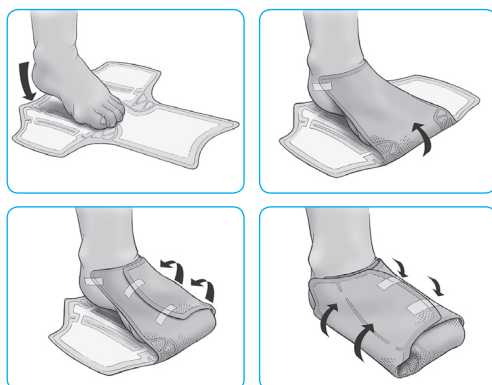

### Contra-indications

Arterial bleeds and heavily bleeding wounds.  
Do not cut Eclipse® Foot.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Ordering information for Eclipse® Foot

Size	Stock code	Pack size	NHS code	PIP code
33cm x 48cm	CR4219	5	EJE086	376-5468



### Features









- ✓ Rapid absorption
- ✓ High capacity
- ✓ Fluid repellent backing
- ✓ Strength and durability
- ✓ Stay dry technology
- ✓ Thin and conforming
- ✓ Single dressing - no patchwork dressings

### Indications

Moderate to heavily exuding wounds:

- ✓ Superficial wounds
- ✓ Pressure ulcers
- ✓ Arterial ulcers
- ✓ Diabetic foot ulcers

### Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize



#### TOP TIP

Eclipse® Foot can be used with additional Eclipse® dressings where the wound area isn't quite big enough to require an Eclipse® Boot dressing.

**Adv**ancis  
Medical  
www.advancis.co.uk